



Opioid Halo™

OPIOID OVERDOSE PREVENTION AND ALERT SYSTEM*

Quick Start Guide



© 2023 Masimo. All rights reserved. 304426/11958A-0323

SYSTEM COMPONENTS




Masimo Chip

- Reusable chip snaps into the sensor and communicates with the Home Medical Hub.



Masimo Halo App

- Displays monitoring status.
- Cloud sends alerts to Home Medical Hub and mobile device.
- Compatible with most mobile devices. (Mobile device not included.)

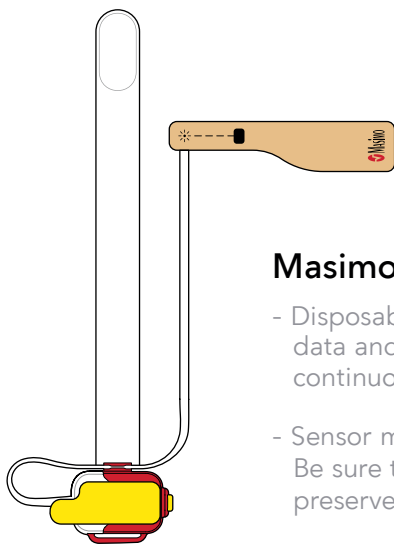

Masimo Sensor


- Disposable sensor provides the monitoring data and lasts up to 96 hours in typical continuous usage.
- Sensor may be removed when not in use. Be sure to remove chip between use to preserve battery life.



Chip Holder

Home Medical Hub

- Collects data from Masimo Sensor.
- Sends data to smartphone app.
- Store the chip in the holder on top of Home Medical Hub.

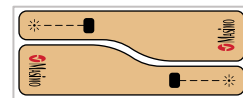

Replacement Tapes


- Additional tapes for replacement as needed.

1 SET UP APP & CONNECT HOME MEDICAL HUB


1. Download the app and create an account.



Masimo HALO

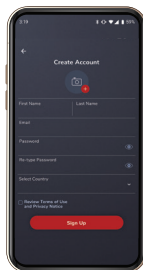

2. Connect Home Medical Hub to power.

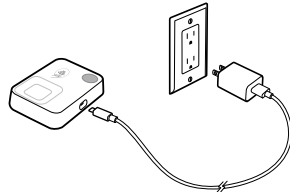

3. Pair Home Medical Hub with phone (Smartphone Bluetooth should be ON).




4. Connect Home Medical Hub to Wifi*. Follow step-by-step instructions on app to set up system.




*Only compatible with 2.4 GHz wireless networks.

2 SET UP SENSOR


1. Remove sensor from pouch.

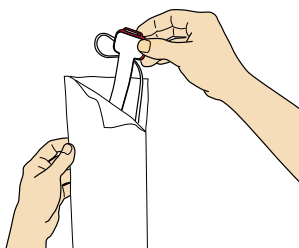


2. Peel off yellow label to activate battery.

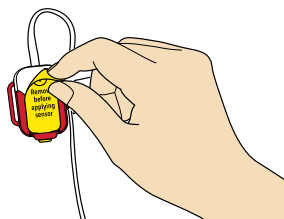

3. Place sensor on wrist as shown and carefully thread the tip of the strap through the open red loop.

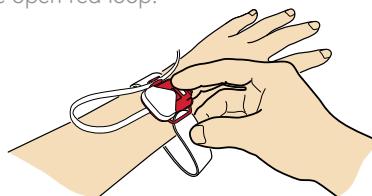


4. Wrap the loose end of the strap around your wrist and press to secure.

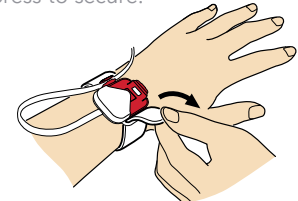


3 APPLY SENSOR


1. Remove part of the plastic film and place star (*) on top of fingernail of ring finger.

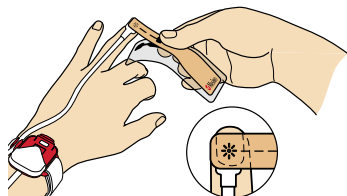


2. Wrap tape around finger so black square (■) is on finger pad directly across from star (*).

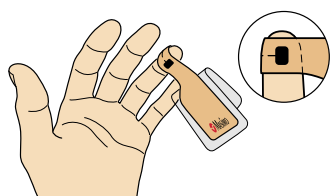


3. Squeeze top and bottom of finger to secure tape.

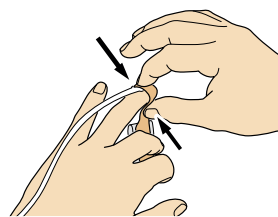

4. Remove plastic film and continue to wrap tape around finger.

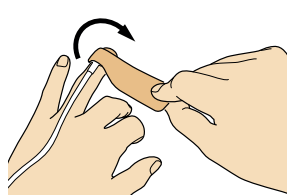


5. Adjust cable to comfortable length.

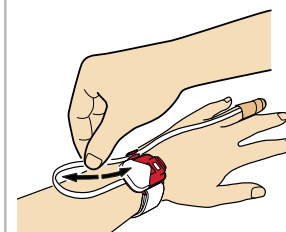


4 PAIR AND INSERT CHIP



1. Hold the chip near the Home Medical Hub and listen for double beep.

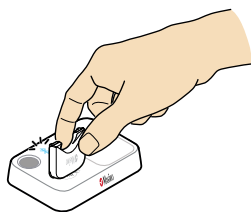


2. Insert the chip into open compartment on the sensor.

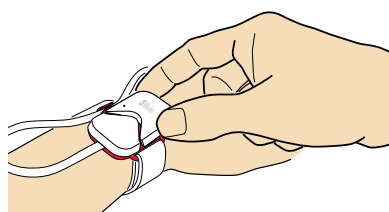

3. Press down on the chip to secure.

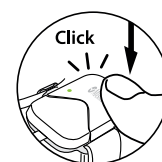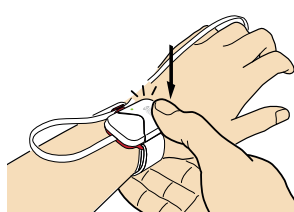


Listen for click to confirm chip placement

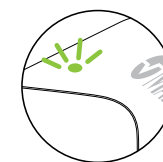

Look for light to confirm chip connection to sensor


5 CONFIRM PAIRING

Once the chip is inserted, the Home Medical Hub Bluetooth Status LED will turn off and the Status LED will turn green. The chip light indicator will turn solid blue.

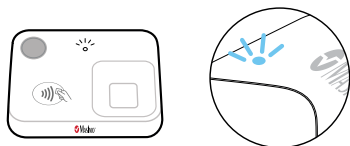

6 SET UP NOTIFICATIONS


1. Add Emergency Contacts from your contact list or add manually.

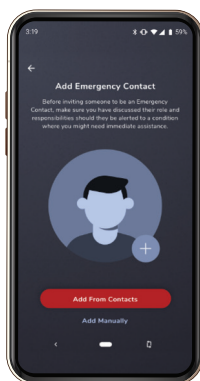

2. Enable emergency dispatch services or choose to opt out.




3. View live data. Green display indicates monitoring has started.

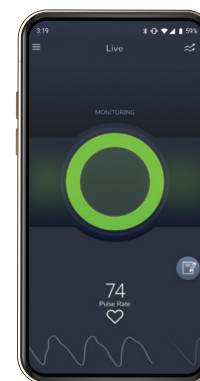


CHANGING THE REPLACEMENT TAPE


1. Remove used tape from sensor.

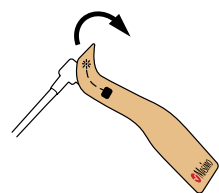


2. Attach new tape in same direction as used tape.

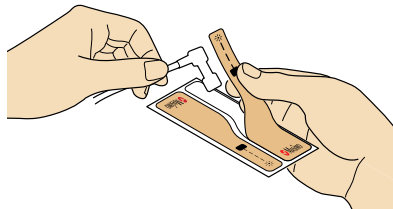

3. Attach sensor to finger.

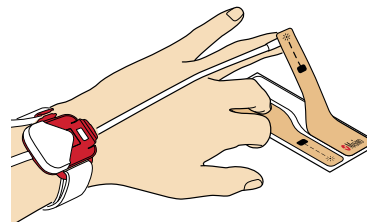

4. Peel the remaining tape off backing.

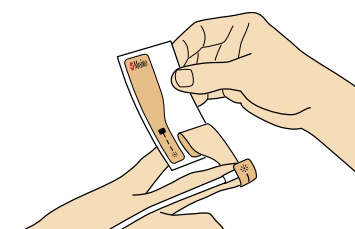

*Opioid Halo continuously monitors certain physiological parameters that are indicative of opioid-induced respiratory depression – a sign of opioid overdose – in order to alert users and their emergency contacts so that action can be taken to avoid adverse consequences of an overdose.

Visit www.masimo.com/support or scan the QR code to:

- see full list of supported devices,
- purchase additional sensors,
- view other resources, including videos and instructions.

