



# MASIMO W1™



## GET TO KNOW YOUR SYSTEM



### MASIMO W1

The watch provides and displays your real-time health data, it will automatically send your data to the Masimo Health App when paired.



### MASIMO HEALTH APP

- › The Masimo Health App collects and displays health data from the watch and other paired devices.
- › Download the app to your Android or iOS device.



### TOUCH SCREEN

- › Swipe left/right or up/down on the screen to interact with the watch.

### MAIN MENU / BACK BUTTON

- › Press once to access the Main Menu.
- › Use as a back button when in submenus.

### LIVE SCREEN

- › From the watch face, press once to see your health data (Live Screen). The screen will return to the watch face after a few moments of inactivity.
- › Press twice to turn off the screen.

## GETTING STARTED


# 1



### SETUP THE MASIMO HEALTH APP

Download the Masimo Health App and follow setup instructions.

# 2


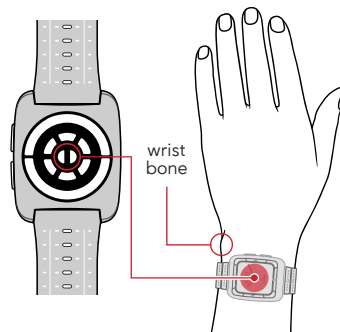


### PUTTING ON YOUR WATCH

Place the watch on the flat part of your wrist, avoiding the wrist bone. Tighten the watch M-band so the watch does not slide on the wrist. Then tighten one or two more notches. To obtain good health data, the watch needs to be worn snug (but not uncomfortably tight).

**TIP:** Ensure the bottom of the watch fully touches your wrist and cannot slide back and forth.

# 3

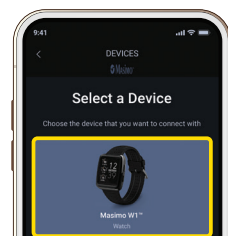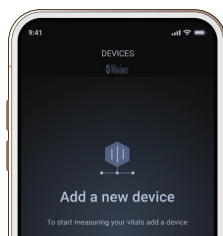


### PAIR YOUR WATCH

In the Masimo Health App, choose **Device** from the Menu. Press the **Add Device** button, then select **Masimo W1** from the list of devices and follow the instructions for pairing the watch to your mobile device.


**TO ACCESS MAIN MENU**




**TO CHECK WATCH STATUS**

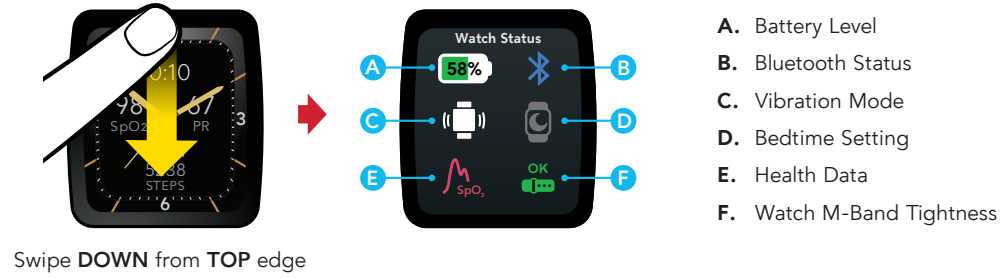

**TO CHECK YOUR HEALTH DATA (LIVE SCREEN)**

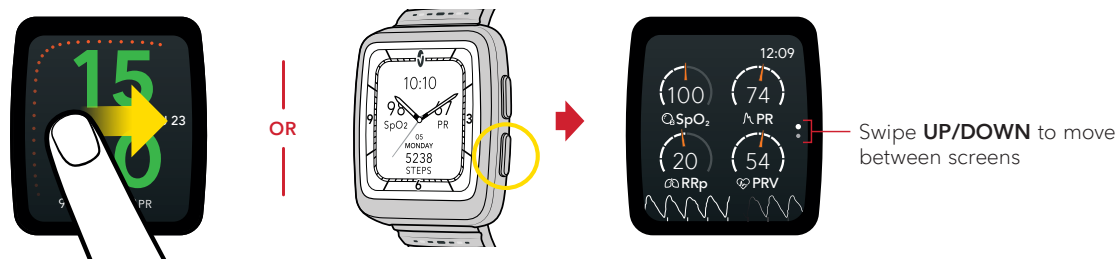

**TO CHARGE YOUR WATCH**

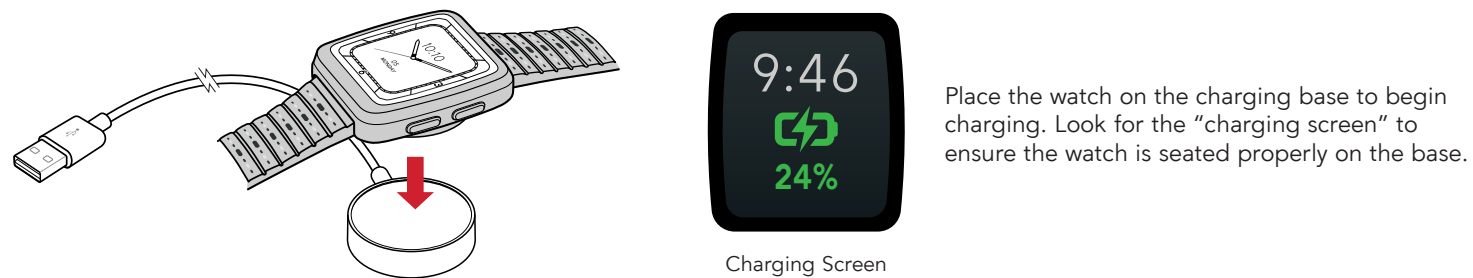

Masimo Corporation, 52 Discovery, Irvine, CA 92618 USA  
www.masimo.com

© 2022 Masimo Corporation. Masimo is a federally registered trademark of Masimo. Masimo W1 is a trademark of Masimo. The Bluetooth word mark and logo are registered trademarks owned by Bluetooth SIG, Inc.

**WARNING:** Do not self-diagnose or self-medicate on the basis of W1 health data. Always consult your doctor.

For more information go to: [www.masimo.com/masimo-w1](http://www.masimo.com/masimo-w1).

Before using this device follow the instructions for use on this website: [www.masimopersonalhealth.com/pages/masimo-w1-support](http://www.masimopersonalhealth.com/pages/masimo-w1-support).