


Masimo SafetyNet®

MightySat Setup Instructions

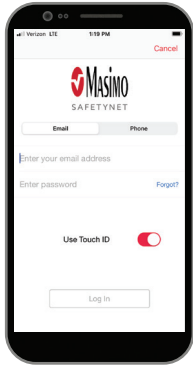

Get to Know Your System

Masimo SafetyNet App

Masimo SafetyNet displays your oxygen saturation measurements. It also allows data updates to your care provider.

WARNING: Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.




MightySat® Rx Fingertip Pulse Oximeter



Getting Started

- 1 Open invite from your care provider.**
In the email click on the link to download the Masimo SafetyNet App. Then open the app and sign up to create your account.
- 2** Once you have created and logged in to your account go to the menu and click on "My Devices". Then select your MightySat device to connect. Follow the on-screen instructions in the app to pair to the MightySat device.

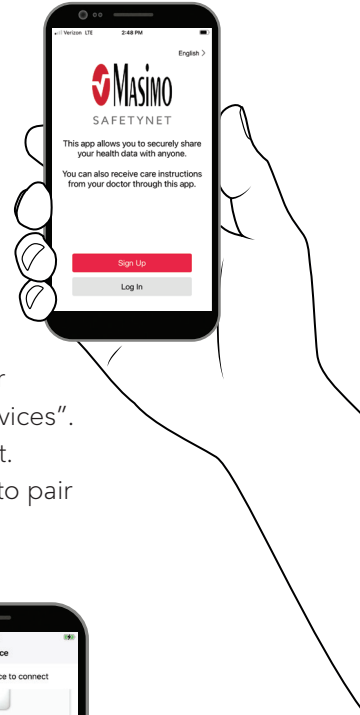


- 3 Take your first measurement**
Follow the instructions in the app showing you how to take a measurement.
(See back for details on taking a measurement.)

For more info go to: mymasimo.com




How to Take a Measurement

- 1 To open the MightySat, squeeze the back portion of the device as shown in the image below.

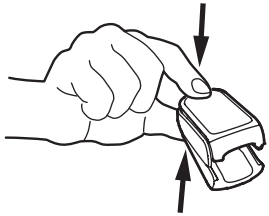


- 2 Once the sensor pads are exposed, insert a finger (non-dominant, ring finger) so the sensor LED is above the fingernail.

NOTE: The display screen of the MightySat should be facing upwards as depicted in the image below.

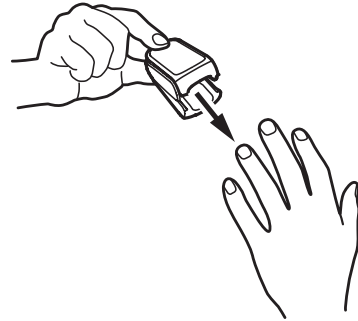


- 3 Once the finger is correctly positioned, gently close the MightySat by releasing the pressure on the back of the device.

NOTE: Ensure the finger is correctly positioned for accurate measurements.

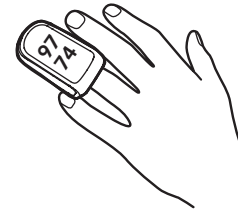


- 4 The tip of the finger should touch the finger stop as shown in the image below.

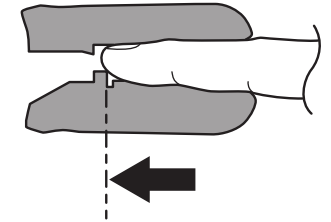

- 5 Once the MightySat is correctly closed on the finger, the MightySat will display readings.

Replacing Batteries

- 1 Place the MightySat so that the display screen is facing downwards.
- 2 Find the battery button on the front of the sensor pad.
- 3 Push lightly on the battery button to release the battery cover and then remove the battery cover.
- 4 Insert two new AAA alkaline batteries and match the orientation labels (+ and -).

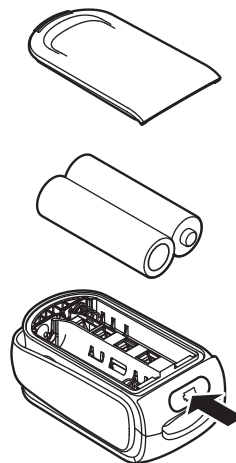

How to Find Your Measurement History

- Tap on "Dashboard" on home screen and then tap on your oxygen saturation chart on the dashboard.
 - Tap on Notification.
- or
- * make sure **Notifications** are enabled.

