


# Root<sup>®</sup> with Radius PCG<sup>™</sup>

Seamless Integration of Tetherless Mainstream Capnography



## Tetherless Solution

Cable-free design and quick Bluetooth<sup>®</sup> pairing facilitate easy movement between care areas, for all patient populations

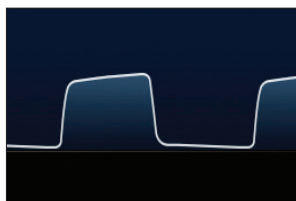

## Increased Visibility

High-resolution, secondary display of large, crisp waveforms and numbers improves visibility of waveform for better interpretation



## Real-time Trending

Clear trend display helps clinicians review progress over time and guide ventilation efforts



## Automated EMR Documentation

Electronic charting of patient data in the EMR via Masimo Patient SafetyNet<sup>™</sup>\* or Iris Gateway<sup>®</sup> simplifies workflow

## Supplemental Capnography Display on Root

Adjust trend display range to tailor monitoring needs for each patient's condition

Up to 96 hours of trend data enables bedside review of changes in a patient's status over time

Clearer, larger waveform display may assist clinicians in identifying wave patterns that suggest airway obstruction or tube dislodgement

Multi-touch screen allows quick adjustments to settings during care

Quick assessment in a dedicated, adjustable window

- Audible and visual alarms for quick identification of alarming parameters
- One-touch Bluetooth menu access for pairing or re-pairing to multiple Root monitors

## Pairing in Three Simple Steps

- 1** From the Root Bluetooth screen, select the Radius PCG Pair button
- 2** On Radius PCG, press the power button twice until the screen displays the Bluetooth symbol
- 3** Select Radius PCG on the Devices Found list on Root and touch the Pair button

## Ordering Information

| ROOT   | RADIUS PCG                                      |
|--|---|
| Root .....9515   | Radius PCG (mmHg) .....9961                     |
| Root with Noninvasive Blood Pressure and Temperature .....9695 | Radius PCG (kPa) .....9962                      |
|  | EMMA® Airway Adapter Adult/Pediatric .....17448 |
|  | EMMA Airway Adapter Infant .....17449           |

\* The use of the trademark Patient SafetyNet is under license from University Health System Consortium.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician. See instructions for use for full prescribing information, including indications, contraindications, warnings, and precautions.

**Masimo U.S.**  
Tel: 1 877 4 Masimo  
info-america@masimo.com

**Masimo International**  
Tel: +41 32 720 1111  
info-international@masimo.com

